

## **Versa Pad Body Strap User Instructions**

The two piece Versa Pad Body Strap is supplied to assist in a wide variety of patient settings. The single use Versa Pad Body Strap is not intended to be used to hold or lift the patient up.

Begin by selecting Strap A (hook fabric) and place Strap A underneath the accessory rail located on the surgical table, making sure the hook fabric fastener is facing upward. Slide the pull end through the slot and tighten the strap firmly to the accessory rail. Repeat this procedure on the other side of the table using Strap B (loop fabric) making sure the loop fabric fastener is facing downward.

Straps A and B are fastened together by placing Strap B over Strap A and connecting the two straps allowing for two point fixation for greater stability.