



O: 801.800.8379
F: 888.209.4417
3390 Orange Street
Riverside, CA 92501

www.safetyingenuity.com

THE VERSA PAD BODY STRAP™

The two-piece Versa Pad Body Strap™ is supplied to assist in a wide variety of patient settings. The body strap is not intended to be used to hold or lift the patient up.

The straps may be placed alongside the surgical table and connected as one unit, utilizing the hook and loop fabric fastener.

The double bar D-Ring design virtually eliminates displacement and slippage of the strap, promoting a more secure connection to the patient's body.

The two-part 3.5 inches wide with a 3-inch hook and loop fabric fastener attached to 35-inch long foam padding, provides soft comfortable support during the surgical procedure.

The two-part Versa Pad Body Strap is fastened together allowing two-point fixation for greater stability. One or multiple body straps may be used to assist in securing the patient. Placement of the straps is selected following the facility's patient positioning protocol.

USER INSTRUCTIONS

Begin by selecting Strap A (hook fabric) and place Strap A underneath the accessory rail located on the surgical table, making sure the hook fabric fastener is facing upward. Slide the pull end through the slot and tighten the strap firmly to the accessory rail. Repeat this procedure on the other side of the table using Strap B (loop fabric) making sure the loop fabric fastener is facing downward.

Straps A and B are fastened together by placing Strap B over Strap A and connecting the two straps allowing for two-point fixation for greater stability.

HOOK STRAP



LOOP STRAP



FOLD D-RING



HOOK & LOOP FRONT



HOOK & LOOP BACK

